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Tastspitze
Pointe de la sonde
Probe Tip
Punta della sonda
Extremo de la sonda

Funktionstaste
Fonctionnement
Operation
Funzionamento
Instrucciones

LCD

Batteriedeckel
Couvercle des piles
Battery Cover
Coperchio batteria
Tapa de la batería

LCD-Anzeige

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
LCD

	< 32,0°C (89,6°F)
	> 42,0°C (107,60°F)
	1. Do not measure when the temperature is not stable. The normal mode will be auto-changed immediately. 2. Short circuit / breakdown 3. Temperature over 50.0°C (122.0°F)
	Power for Operation Indicator
	Lower Battery Indicator

Measuring Procedures

It is very important that probe tip be placed in the right location, as instructed and illustrated on illustrated.

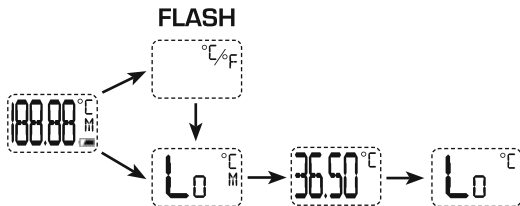1. Turn on the thermometer by pressing the button.
2. LCD will show some characters as illustrated below.

3. Wait for 2 beeper sounds and the unit is now ready to use.
4. In selecting the °C or °F value by pressing the button for 2 Seconds when °C or °F symbol is flashing.

Maintenance and Storage

The unit is not waterproof / water resistance and never submerge into water or other form of liquid. You may use moistened cotton or alcohol swab with 70 % isopropyl in wiping the tip and probe before or after use. While only use a soft dry cloth in cleaning the plastic body casing. The following helpful tips will enable you to maintain and secure optimum result, as well prolong the life of the unit.

1. Always remove the battery from the unit when not to be used in long time and store the unit in a room temperature
2. Do not expose the unit to a direct sunlight or high temperature and humidity.
3. Do not drop the unit as it is not a shock proof and may cause damage with electronic parts.
4. Keep out the unit from the reach of children.
5. Use the unit per illustrated and indicated instructions with this manual.
6. Keep a distance away from any electro magnetic transmitters or appliances that may cause disturbance, as well inaccurate measurement of the unit.
7. The unit is fully calibrated and has been carefully tested.
8. In case of malfunctions, refrain from doing any repair and bring the unit back to where it's purchased or only to authorized agent.

Removing and Installing the Batteries

Please refer to hereunder illustrations and follow the instructions.

- a. Press the snap and slide down the battery cover from the unit.
- b. With only a little pressure, remove the batteries from the compartment.
- c. With emphasis for the right polarity indicated in the battery compartment, install a new set of batteries.
- d. Slide back gently the battery cover.
- e. Only use the right type of battery, which is of AAA size x 2 pieces.
- f. Replace all batteries whenever almost exhausted or symbol shows on the LCD window.
- g. Discarded batteries should be disposed properly.

Warning

- Before you start using this unit, please read this operation manual thoroughly for the correct methods of usage.
- Do not drop or disassemble the unit.
- Avoid exposing the thermometer to direct sunlight, high temperature, moisture or dirt.
- If the unit will not be used for a long time, it is advisable that the battery be removed and stored from the reach for children.
- If the unit becomes contaminated, clean it with a damp cloth.

Warranty

This thermometer carries a warranty of 1 year from the date of purchase. The distributor reserves the right either to repair or replace without charge necessary parts to correct defects in the materials or workmanship.

The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, material or disassembly by unauthorized individuals.

Disposal in EU countries

Do not dispose of the appliance with domestic refuse. As part of the EU Directive governing the disposal of electric and electronic appliances, the appliance is accepted free of charge by local waste collection points or recycling centers. Correct disposal will ensure environment protection and prevent a potential harmful impact on people and the environment.

Product name	Animal Electronic Digital Thermometer
Model Number	21124
Measurement Range	32.00°C ~ 42.00 °C (89.60°F ~ 107.60°F)
Temperature Scales	°C or °F (user selectable) Press ON/OFF button more than 2-second to set up LCD display °C/°F
Display Resolution	0.01°C or 0.01°F
Accuracy	± 0.10°C between 32.00°C to 42.00°C ± 0.20°F between 89.60°F to 107.60°F
Measurement Error	LCD shows "Hi" when > 42.00°C
Error Operation	An "Err" symbol will appear on LCD display, it will BiBiBi sound and shut off immediately with hurried BiBi sound.
Completed Measurement	4 long beeper sounds
Display Mode	Rectal
Measurement Time	Water bath (standard device): about 10-second Rectal: about 15-second
Auto Shut-Off Time	30 seconds after using or remained idle
Memory Recall	Last temperature
Fever Alarm	Rapid and successive beeper sounds in 4 seconds
Operating Temperature /	5.0°C ~ 45.0°C (41.0°F ~ 113.0°F)
Humidity	30% ~ 80 % RH
Storage Temperature /	-10.0°C ~ 60.0°C (14.0°F ~ 140.0°F)
Humidity	10 % to 80 % RH
Battery Type	2 x 1.5V (AAA) DC
Battery Life	Approximately 5,000 times for measurement
LCD Dimension	18.5mm (L) x 18.5mm (W)
Product Dimension	22.5 x 3.7 x 7.0 cm
Weight	69.0 grams

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Albert Kerbl GmbH
Kerbl Austria Handels GmbH